SinusInhaler

Personal Steam Inhaler

Use and Care Manual

- ♥ Comforting treatment in minutes
- ♥ Adjustable steam output

The Vicks Steam Inhaler provides a steady flow of warm, moist air into the nose and throat for temporary relief. If you have any questions about the operation of this Steam Inhaler, call our toll-free Consumer Relations line at 1-800-VAPOR-1-2 or visit: www.VicksHumidifiers.com/contact-us.

IMPORTANT!
READ AND SAVE THESE INSTRUCTIONS

Model
VIH200 Series

⚠️ Important Safety Instructions

READ AND SAVE THESE IMPORTANT SAFETY INSTRUCTIONS BEFORE USING THIS APPLIANCE

⚠️ WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

⚠️ DANGER To reduce the risk of electric shock:

- 1. ALWAYS** turn off and unplug this product immediately after using.
- 2. DO NOT** use while bathing.
- 3. DO NOT** place or store product where it can fall or be pulled into a tub or sink. The appliance should always be placed on a firm, flat, waterproof and heat resistant surface. Be sure the appliance is in a stable position and the power cord is out of the way to prevent the appliance from being overturned.
- 4. DO NOT** place in or drop into water or other liquid.
- 5. DO NOT** reach for a product that has fallen into water. Unplug immediately.

⚠️ WARNING To reduce the risk of burns, electric shock, fire, or injury to persons:

- 6.** The product should never be left unattended when plugged in.
- 7.** Close supervision is necessary when this product is used by, on, or near those who are physically challenged. **Keep out of reach of children.**
- 8.** Use this product only for its intended use as described in this manual. **DO NOT** use attachments not recommended by the manufacturer.
- 9. NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged or dropped into water.
- 10.** Keep the cord away from heated surfaces.
- 11. ⚠️ CAUTION: NEVER** move or disassemble the appliance while in use. It can spill hot water if tilted, shaken, or overturned which may lead to injury or burns.
- 12. DO NOT** use appliance while sleepy or drowsy.

- 13. DO NOT** use in area where aerosol products are being used or where oxygen is being administered.
- 14. NEVER** block the air openings and vents of the product or place it on a soft surface, such as a bed or couch, where the air openings and vents may be blocked. Keep the air openings free of lint, hair, and other materials.
- 15. NEVER** drop or insert any object into any opening or vent.
- 16.** Unplug this product before filling. Fill (reservoir) with water only unless otherwise specified by manufacturer. **DO NOT** overfill. **DO NOT** fill with mask installed. Only pour water directly into Water Reservoir.
- 17.** The appliance has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT** attempt to defeat this safety feature.
- 18. ⚠️ CAUTION:** To avoid fire or electric shock hazard, plug the appliance directly into a 120V AC electrical outlet. **DO NOT** use an extension cord or a power strip.
- 19.** The appliance should always be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug the appliance before moving. Plug and unplug appliance with dry hands. **NEVER** pull by the cord to unplug.
- 20. DO NOT** operate the appliance without water.
- 21. DO NOT** lift the appliance by the Mask. Lift only by the Base.
- 22.** Appliance requires regular cleaning. Refer to and follow cleaning instructions.
- 23. NEVER** add any essential oils, medications or inhalant liquids to the Water Reservoir, such as Vicks VapoSteam.

- 24. DO NOT** touch skin with exposed Vicks VapoPads or place on furniture, fabrics, bedding or plastics.
- 25. DO NOT** operate outdoors. This appliance is intended for indoor use.

- 26. DO NOT** attempt to repair or adjust any electrical or mechanical functions on this appliance. Doing so will void your warranty.
- 27.** For residential use only.

SAVE THESE INSTRUCTIONS

Accessories

Vicks VapoPads


(One Menthol VapoPad Sample Included)

Vicks VapoPads are designed to work with your Steam Inhaler to provide soothing vapors to complement your steam therapy efforts. The Steam Inhaler accepts up to two Vicks VapoPads. Enjoy up to 8 hours of soothing comfort per pad. Menthol scent: VSP-19 Series

Vicks Sinus Inhaler

1. Mask
2. VapoPad Tray
3. Top Housing
4. Steam Disk Controls
5. Base
6. Water Reservoir
7. Power Level Switch
8. Vents
9. Heating Element

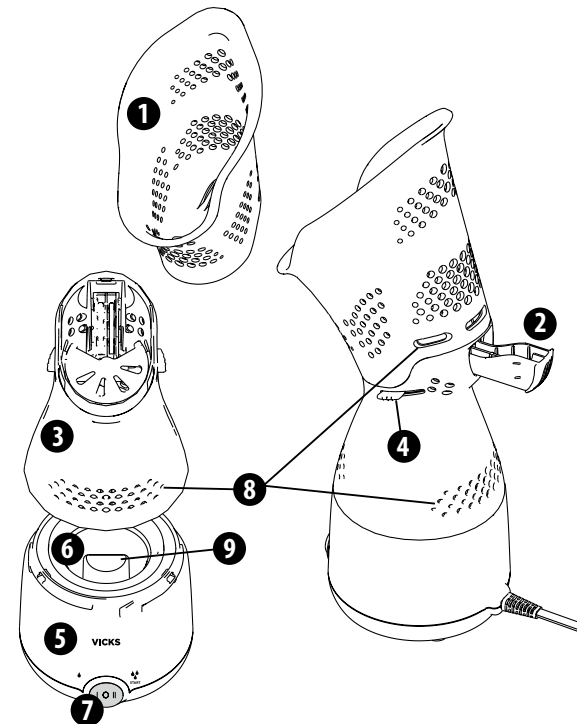The Vicks Steam Inhaler provides a steady flow of warm, moist air directly to the nose and throat for temporary relief.







Using Your Personal Steam Inhaler

1. Fully unravel cord. Failure to do so could cause overheating and possible fire hazard. **DO NOT** use with extension cords or power strip. Place Inhaler on a flat, stable surface for use.
2. Unlock the Top Housing from the Base by twisting the Top Housing counter clockwise. When fully unlocked, the arrow ▼ on the Top Housing will align with the unlock symbol on the Base. Lift the Top Housing straight up and set aside.
3. Fill Water Reservoir with up to 2.5 tablespoons (40 mL) of cool tap water. **DO NOT** fill above Heating Element. Use with water only. **DO NOT** add any essential oils, medication or inhalant liquids such as Vicks VapoSteam to the Water Reservoir. An unsafe condition may result if anything is added to the water.
4. Place Top Housing on the Base and twist clockwise to lock. When fully locked, you will feel a click and the arrow ▼ on the Top Housing will align with the lock symbol on Base.
5. Place Mask on Top Housing and gently push down to ensure Mask is secure.

CAUTION: NEVER move, lift, shake or tilt the Inhaler while it is in use or while it contains hot water. **DO NOT** remove the Mask or Top Housing after turning ON the Inhaler. Steam is produced by boiling water; tilting or shaking unit can cause water to spill and possible burns.

CAUTION: DO NOT block any vents.

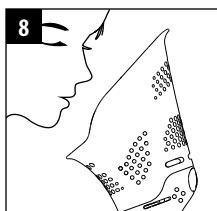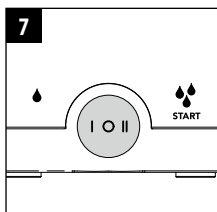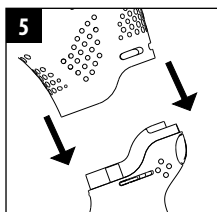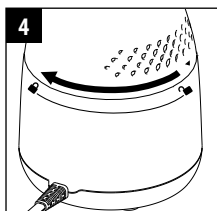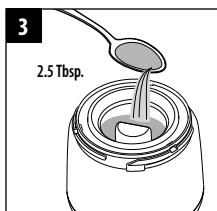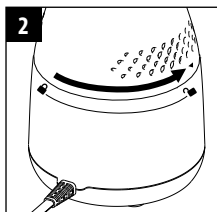6. Connect cord to electrical outlet. **DO NOT** use with extension cord or power strip.
7. Turn power switch to High/Start  to start. The Indicator Light will illuminate. The Steam Inhaler will take approximately 3 minutes to begin making steam on high setting. Use caution, the Water Reservoir and mist will become hot.
8. Place your face over the Mask. **DO NOT** rest your face on the Mask. Breathe normally through your nose and mouth, inhaling steam into your nasal and throat passages. Continue treatment for five to fifteen minutes.

Using Your Personal Steam Inhaler (continued)



9. Adjusting steam output and intensity:

A: Steam Output: For More Steam output and faster steam, use the High Power Setting . For less steam output switch the power switch to the Low Power Setting .

B: Steam Intensity: For more steam flow into the mask, turn the steam disk control clockwise. Turn steam disk counter clockwise to reduce steam intensity.

10. When treatment is complete, move Power Switch to the OFF position. The Indicator Light will go out. Disconnect plug from electrical outlet. Allow the appliance to cool for at least 20 minutes before moving or refilling it (see Care and Cleaning section). **DO NOT** operate Steam Inhaler when empty.

NOTE: Steam Inhaler will automatically shut-off when water runs out. If this happens, move Power Switch to OFF position and unplug unit.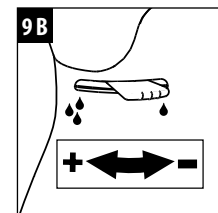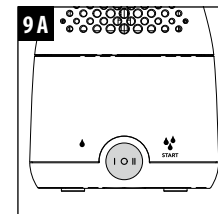

Using Vicks VapoPads




Your Vicks Steam Inhaler can be used with Vicks VapoPads, to provide up to 8 hours of soothing vapors. A free sample of the menthol VapoPad is included with your device.

To use your unit with scent pads, ensure that the unit is OFF and follow these instructions:

1. Gently slide out the VapoPad Tray from the back of the Top Housing. **NOTE:** Only slide Tray out. **DO NOT** attempt to detach the Tray from the Top Housing.
2. Open VapoPad sachet by tearing notch on pad bag. **DO NOT** touch pad with hands. If pad is touched, **DO NOT** rub face or eyes as it may cause irritation.
3. Insert Vicks VapoPad with angled end at the front of the tray. Up to two scent pads can be inserted, one in each slot of the VapoPad Tray. Push VapoPad Tray in to close.
4. When the VapoPad is dried and no longer providing scent, discard.

NOTE: DO NOT operate Steam Inhaler if VapoPad Tray is missing or not secured.

This Steam Inhaler (with or without Vicks VapoPads) is not recommended for use by children.

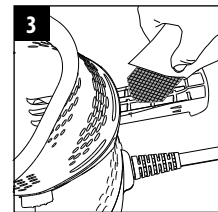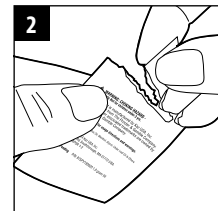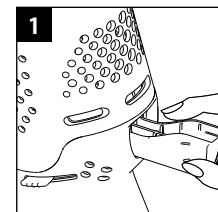Using Vicks VapoPads (continued)

VapoPads Precautions:

- Not recommended for infants under 10 lbs (4.5 kg).
- For children between 10-22 lbs (4.5-10 kg) **DO NOT** use more than 2 pads in a 24 hour period.

Scent Pad Ingredients: VSP-19 Series: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others.

Care and Cleaning

 **CAUTION: ALWAYS** disconnect the plug from the electrical outlet and let the Steam Inhaler cool down for at least 20 minutes before cleaning.

After Each Use

1. Remove Mask and Top Housing and wipe dry with a clean cloth or paper towel.
2. If any water is left in the Water Reservoir, pour water out and wipe Reservoir and Base with a dry, clean cloth or paper towel. **NEVER** immerse the Base in water or any other liquid.
3. Replace Top Housing and Mask; store in a cool, dry place.

Weekly Maintenance

1. Remove Mask and Top Housing.
2. Pour 2.5 tablespoons (40mL) of undiluted distilled white vinegar into the Water Reservoir and let sit for 10 minutes.
3. Empty the distilled white vinegar and rinse out Water Reservoir with fresh, clean tap water. Use caution to not let water get into the electrical areas.
4. Dry the Water Reservoir and Base with a clean cloth or paper towel.
5. Wash Mask with mild soap and warm water then carefully rinse off all soap with fresh, clean tap water. Dry with a clean cloth or paper towel.
6. Replace Top Housing and Mask. Store in a cool, dry place.
7. To clean the VapoPad Tray, wet the tip of a cotton swab and wipe slots.

NOTE: The Top Housing is dishwasher safe. It may be cleaned in the Top Rack of the dishwasher or by hand as noted in the instructions. **DO NOT** place in the bottom rack of dishwasher. **DO NOT** attempt to disassemble the Top Housing. The Heating Cup should **NEVER** be put in the dishwasher.

Care and Cleaning (continued)

After Long Periods of Non-Use

If you haven't used your Steam Inhaler for two weeks or longer, follow the Weekly Maintenance Instructions before the next use.

Condition of Sale

As a condition of sale, the purchaser assumes responsibility for the proper care and use of this appliance in accordance with our printed instructions. The purchaser or user must judge for himself or herself when to use it and length of use.

Specifications

Electrical rating: 130W 120V, 60hz

Consumer Relations

We are here to help.

Call us toll-free at: 1-800-VAPOR-1-2 (1-800-827-6712)

Visit: www.VicksHumidifiers.com/contact-us

Please be sure to specify Model Number VIH200.

1 Year Limited Warranty

READ ALL INSTRUCTIONS BEFORE ATTEMPTING TO USE THIS PRODUCT.

Please retain the receipt as proof of and date of purchase. The receipt must be presented when making any claim within the relevant warranty period. Any claim under the warranty will not be valid without a proof of purchase. Your device is warranted for 1 year (1 year) from date of delivery.

- A.** This 1 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty. **KAZ IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.** Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction. This warranty applies only to the original purchaser of this product from the original date of delivery.
- B.** At its option, Kaz will repair or replace this product if it is found to be defective in material or workmanship.
- C.** This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.
- D.** This warranty does not cover the filters, pre-filters, UV bulbs or other accessories (if included) except for material or workmanship defects.

Call us toll-free at 1-800-VAPOR-1-2 (1-800-827-6712) or visit:

www.VicksHumidifiers.com/contact-us

Please be sure to specify Model Number VIH200.

NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE PRODUCT TO THE ORIGINAL PLACE OF PURCHASE. DO NOT ATTEMPT TO OPEN THE BASE. DOING SO MAY VOID YOUR WARRANTY AND CAUSE DAMAGE TO THE PRODUCT OR PERSONAL INJURY.

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© 2026 All Rights Reserved.
Certain trademarks used under license from The Procter & Gamble Company or its affiliates.
Distributed by: Kaz USA, Inc., a Helen of Troy Company
Distribuido por: Kaz USA, Inc., una empresa de Helen of Troy